


# EasyTrack FS | Voyager

## AVAILABLE SLINGS



**HAMMOCK SLING**  
For most transfers.  
Head-rest incorporated.  
Suitable for most individuals.  
Excellent for bathing.

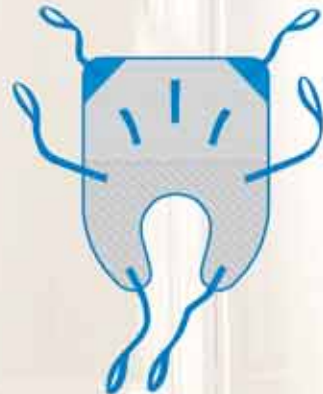

**QUICK-FIT SLING**  
For general transfers.  
Perfectly suited for stocky  
or obese patients with  
strong hips or large thighs.



**HYGIENIC SLING**  
Ideal for transferring to the  
toilet. It also permits an easy  
way to change, dress or  
undress the individual.

	SMALL	MEDIUM	LARGE
Suggested weight	45-100 lbs (20-45 kg)	100-200 lbs (45-90 kg)	200-600 lbs (90-272 kg)
Suggested height	4'-5'11" (120-180 cm)	5'-5'11" (152-180 cm)	6' + (181 cm +)
Hammock	THA-S	THA-M	THA-L
Hammock 6	THA6-S	THA6-M	THA6-L
Quick-fit	TIR-S	TIR-M	TIR-L
Hygienic	THY-S	THY-M	THY-L



**EasyTrack FS system # 9720010 Includes :**  
Extendable rail with trolley, 2 telescoping posts,  
2 foot plates.

The system has **5** easy-to-assemble components.

Optional protective carrying bags makes the system easy to transport.



**Bag Kit # 002.17010 Includes :**

- 2 Post Bags: # 002.17000
- 1 Base Bag: # 002.17100
- 1 Rail Bag: # 002.17250

Bags also available individually



**BHM**  
*Because Health Matters*

Canada  
2001 Tanguay, Magog  
Québec J1X 5Y5  
5155 Spectrum Way, Suite #33  
Mississauga, Ontario L4W 5A1  
Europe  
PO Box 28993, London SW14 8YL  
UNITED KINGDOM  
☎ 819 868-0441  
☎ 819 868-2249  
✉ bhm@bhm-medical.com  
[www.bhm-medical.com](http://www.bhm-medical.com)

BHM Medical Inc. Reserves the right to change or discontinue any specifications, design, features, model or accessories shown without notice.

Printed in Canada 009.17005



# EasyTrack FS

*Featuring the Voyager portable lift*

A true portable lift and transfer system for the home that's easy to transport and quick to assemble.



**BHM**  
*Because Health Matters*

# EasyTrack FS

The Easytrack FS is a free-standing, versatile solution for all types of point-to-point transfers - wheelchair to bed, bed to chair, even in the bathroom, anywhere there is a lift and transfer need.

The Easytrack FS has a safe working load of 440 pounds, or 200 kilograms. That's exceptional for a portable product making it suitable for use in most any home.

This equipment was designed to allow those with a mobility impairment to continue to live at home and enjoy many of your normal activities.



1

The light weight aluminium bases feature adjustable levellers to ensure that the unit remains perfectly stable and secure when assembled.



2

The posts have been designed to be self-aligning. Components only fit together one way to prevent any assembly errors and ensure the safety of the system.



3

The rail can be extended up to a width of 115 inches (3 meters). That is more than enough to span a king size bed.



4

To insert the top rail, the locking clamps on both posts are opened - pushing down on the security catch unlocks and opens the clamp and securely fasten the top rail in place.



5

The system can be adjusted to a height of 94.5 inches (2.4 meters). That is just short of the 8 foot ceiling found in most homes.



6

The telescoping posts include height indicators that allow you to adjust both posts for a perfectly level top rail. A special one-way locking mechanism ensures that the post cannot retract on its own.



7

The next step in readying your partner for transfer is to position the sling around him and secure the sling straps to the lift\*.



8

A BHM portable, battery-powered rechargeable lift\* is used to gently lift and assist you in the safe and secure transfer of your loved one.

**BHM Medical...because health matters**

\* Lift shown is sold separately and not included with Easytrack FS kit.